



MYCO
MEDICAL

TheSharpChoice®

NMSDC Certified Diversity Supplier

RELI® BLUNT FILL NEEDLE

Instructions for Use

NOT FOR SKIN INJECTION!

STEP 1:

Secure hub of fill needle to syringe by twisting clockwise. Remove cap and save.

NOTE: Disinfect vial stopper with alcohol prior to piercing.



STEP 2:

Puncture the center of the vial stopper at 90° to minimize risk of catching internal wall of vial stopper.

NOTE: Remember to aspirate into the syringe an amount of air equal to the volume in the vial to be removed. Inject air into the vial prior to medication removal.



STEP 3:

After filling, passively recap hub of fill needle using passive one-handed recapping technique.

NOTE: Syringe is now ready for use with IV bags or IV Systems.



STEP 4:

Detach fill needle from syringe by grasping at the base of the needle and twisting counter-clockwise.

NOTE: Syringe is now ready for administration via luer-activated valves.



STEP 5:

Discard fill needle in approved sharps container.



Reli® Blunt Fill
Needles (Sterile)



To order, learn more or receive free evaluation samples:

Please call MYCO Medical at (800) 454-6926 or email us at customerservice@mycomedical.com

Reli® Blunt Fill Needles (Sterile)

MYCO CODE	GAUGE	LENGTH	UNIT OF MEASURE
BFN18G101	18G	1"	100 ea/box
BFN18G151	18G	1.5"	100 ea/box
BFN20G101	20G	1"	100 ea/box